

“BRUX” Full Contour Solid Zirconia Crown Adjustment

- Adjust Brux restorations using water and air spray to keep the restoration cool. To avoid microfractures, use a fine grit diamond. If using air only, use the lightest touch possible when making adjustments. A football shaped bur is the most effective for occlusal and lingual surfaces (on anterior teeth); a tapered bur is the ideal choice for buccal and lingual surfaces.
- Polish Brux Solid Zirconia restorations with the porcelain polishing system of your choice. Diamond impregnated rubber cups and points work well.